MINI x TRES

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Implantes BEGO SEMADOS® MINI

*BEGO SEMADOS® MINI IMPLANTS
Economically solutions for
Bar-, Ball- and Single-Tooth-Treatments*

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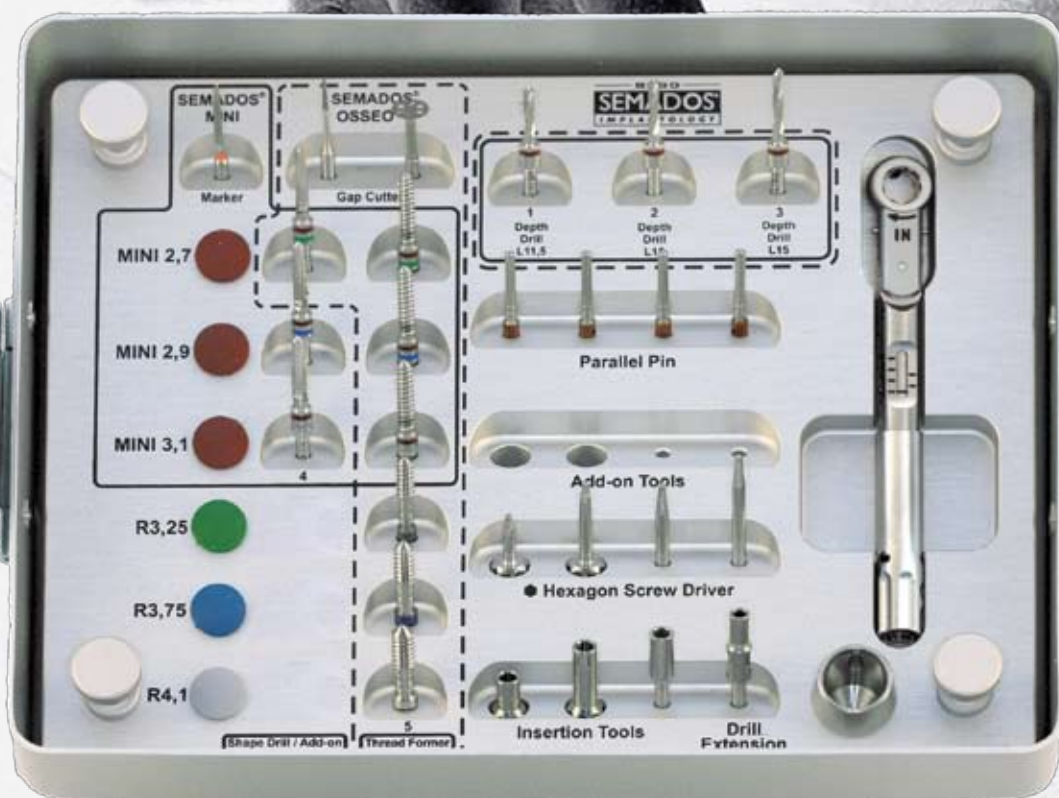
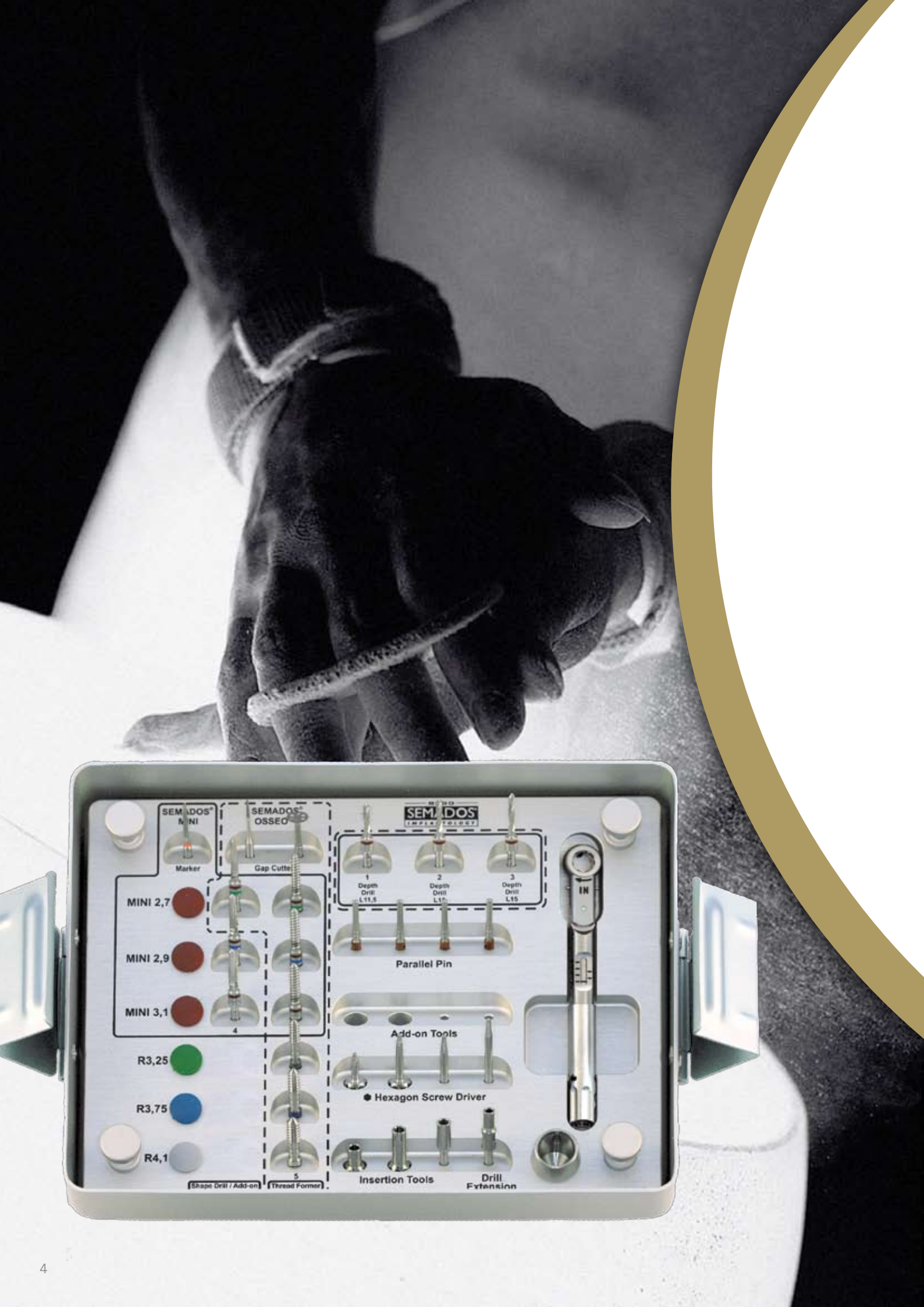
If an edentulous jaw is left untreated for a prolonged period, this will result in the resorption of the alveolar process. Over time, there is a serious horizontal loss of substance in the anterior maxilla and mandible, which can lead to a very narrow alveolar ridge. Treating such cases with implants presents the dental practice with considerable problems. Augmentation techniques, including autogenous bone block grafts, are often recommended to the patient affected. If, instead, the narrow crestal bone regions are simply levelled out by milling, this leads to a significant height loss and consequently soft tissue problems in most cases.

Often such compensation techniques entail referral to an oral and maxillofacial surgeon. Yet many patients are put off by the prospect of these invasive measures. To solve problems of this kind, experienced implantologists have developed a minimally invasive and inexpensive alternative treatment concept in collaboration with a working group from BEGO Implant Systems, the Gießen-Marburg university clinic (Prof. Dr. Howaldt and team) and the IZI (Institute of Dental Implantology), under the expert coordination of Dr. Dr. Roland Streckbein and Dr. Dr. Philipp Streckbein.

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An innovative concept – bone condensing/spreading

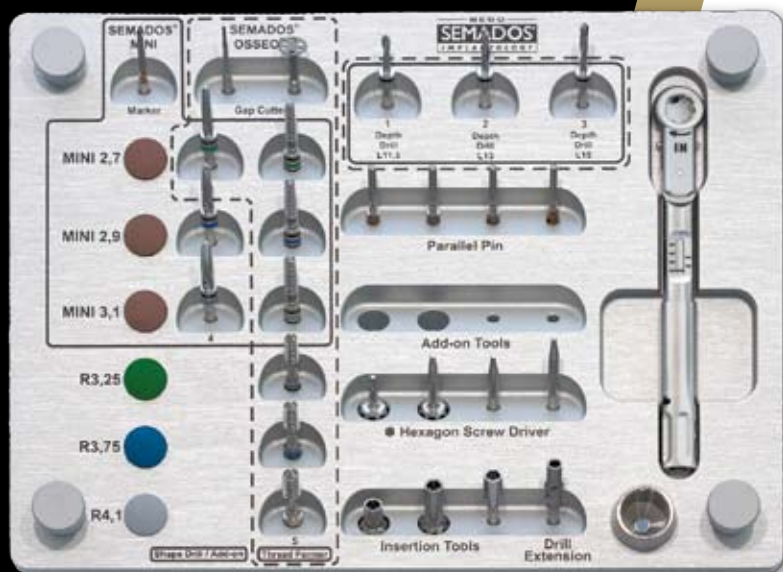
Treatment with BEGO Semados® Mini Implants is made possible by a bone management concept designed to conserve bone substance. The nonablative thread former is used to produce small pilot holes manually, at the same time compressing the bone mass. The asymmetric thread flanks, with the screw's overall conical form, gently spread the bone with lateral displacement. In this way, narrow alveolar

ridges can also be treated without any loss of buccal bone lamellae.

Minimum diameter for minimally invasive surgery

For the narrow edentulous alveolar process, a combined surgical and prosthetic treatment is now successfully established. A prefabricated non-precious alloy bar system (BEGO Wirobond® MI) has been developed, which is tailored BEGO Semados® Mini implants (dia. 2.7, 2.9, and 3.1 mm). The cobalt-chrome alloy permits very slender but nevertheless high-strength bar designs. The technique is simple. Prefabricated parts are cut to length and joined by laser welding. On the day of implant placement, primary splinting with a high-quality bar frame can be carried out at no great expense. Immediate loading then presents no problem.

A ball attachment restoration for two implants is available for treating edentulous lower and upper jaw conditions. For optimal load bearing, implants with a diameter of 2.9 mm and 3.1 mm should be used. The conical Wirobond® MI abutment should be used to replace the upper lateral incisors and the lower incisors (32/31/41/42). The use of a diameter less than 2.9 mm is not permitted.

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*Indication for single tooth restorations with the BEGO Semados® Mini-Line: 12,22,32-42

1 Fresado piloto con fresa triangular Pilot drilling with initial drill

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Gap cutter – Disc for initial splitting of narrow alveolar ridges.

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Gap cutter – Burr for deep splitting of alveolar ridges.

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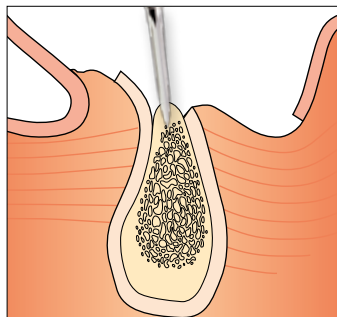
Final depth drilling. Check with paralleling post.

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
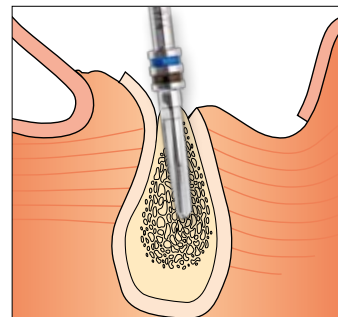
*Conical enlargement
of the implant bed.*

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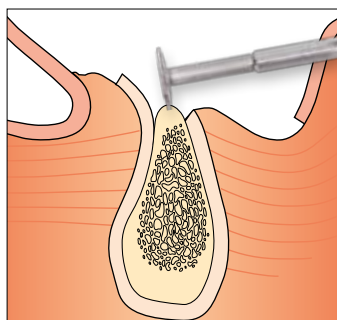
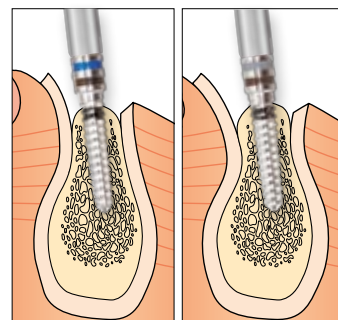
Forming (condensing) a thread down to the floor of the implantation site and spreading the narrow alveolar ridge to the desired implant diameter. Can be performed manually or with a handpiece.

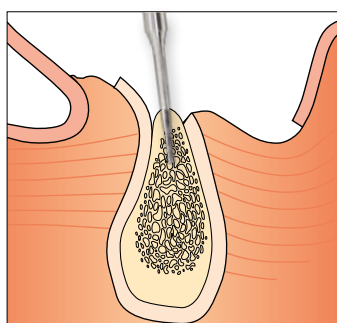⑤ Inserción manual o mecánica del implante *Implant insertion – manually or with a handpiece*

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
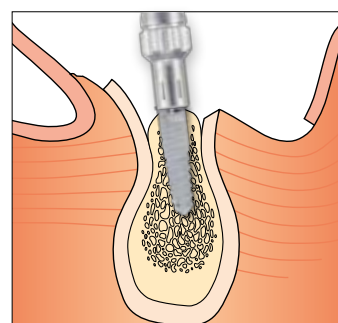
Insert the implant into the bone with the ratchet or insertion tool used with a handpiece (polished section remains above the bone).

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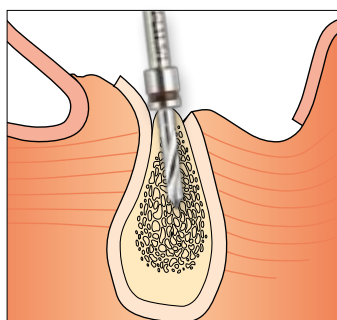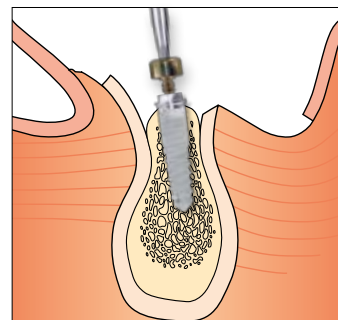
Pick up the cover screw with the hexagon screwdriver, insert it into the implant and hand-tighten.

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2

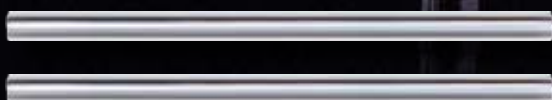
BARRA

- *Treatment of narrow, edentulous maxillae and mandibles, particularly in the anterior region*
- *No augmentation surgery required*
- *Minimally invasive procedure using diameter-reduced BEGO Semados® Mini implants*
- *Combination of bone spreading and bone condensing with a coordinated system of tools*
- *High primary stability of the inserted implants*
- *Immediate restoration of the implants by means of a non-precious alloy round bar system for primary splinting*
- *Simple integraton of prefabricated matrices, e.g., into existing full dentures.*
- *Excellent wearing comfort with dentoalveolar design of the restoration*
- *Simple and inexpensive implementation using standard prefabricated parts*
- *The splinted bar frame can be placed without any difficulty with abutment divergences of up to 15° per Implant*

COMPONENTS FOR BAR RESTORATIONS

For treating edentulous narrow mandibular and maxillary alveolar processes (with 4 implants) you will need:

- 1 4 BEGO Semados® Mini implants (including implant cover screw) (choice of Ø 2.7/ 2.9/ 3.1)
- 2 4 Mini open tray impressions (including model analog), Art. no. 56959
- 3 1 Wirobond® MI Mini bar abutment for 4 implants (including 4 Wirobond® MI bar abutments, 4 prosthesis screws, 4 technician screws as well as 2 Wirobond® MI round bars 40mm), Art. no. 56963
- 4 1 round bar system "PRECI-HORIX", Art.-no. 55775

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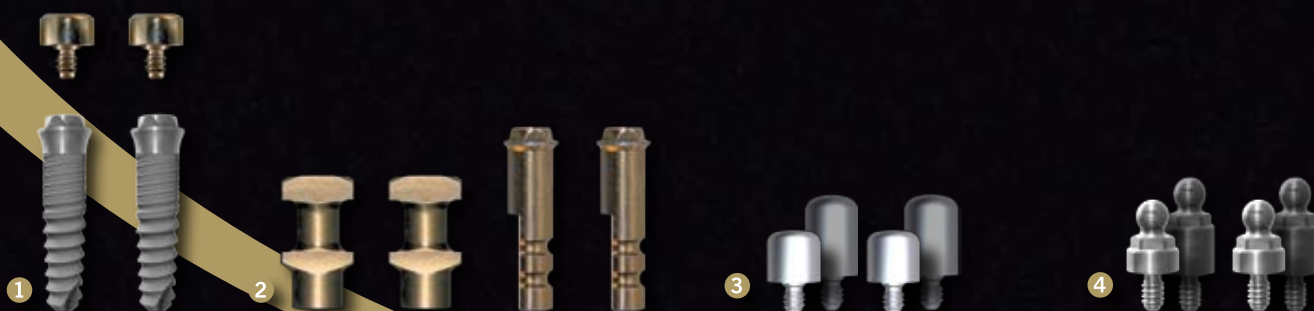
TE ANCLAJES DE BOLA

- *Ball attachment restorations on only two implants for diameters 2.9 and 3.1*
- *Less augmentation surgery required*
- *Combination of bone spreading and bone condensing using a coordinated system of tools*
- *Asymmetric thread flanks ensure extremely high primary stability of the inserted implants*

COMPONENTS FOR BALL ATTACHMENT RESTORATIONS

For treating edentulous narrow mandibular and maxillary alveolar processes (with 2 implants) you will need:

- 1 2 BEGO Semados® Mini implants (including implant cover screw) (choice of Ø 2.9/ 3.1)
- 2 2 Mini open tray impressions (including model analog), Art. no. 56959
- 3 2 Mini healing posts (choice of 3 or 4 mm length)
- 4 Mini ball attachments (choice of 1.5 or 4 mm height)
- 5 Mini ball attachment laboratory set, Art. no. 57085
(including 2 x PRECI-CLIX housings; 2 x PRECI-CLIX duplication auxiliaries;
2 x PRECI-CLIX female parts, yellow (normal retention); 2 x PRECI-CLIX spacer)

Optionally available:

- 6 PRECI-CLIX female part, white, Art. no. 57087
PRECI-CLIX female part, red, Art. no. 57088

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TREATMENTS PARA

- *Restoration of small anterior gaps (12, 22, 32-42)*
- *Less augmentation surgery required*
- *Combination of bone spreading and bone condensing using a coordinated system of tools*
- *Asymmetric thread flanks ensure maximum primary stability after insertion*

COMPONENTS FOR SINGLE TOOTH RESTORATIONS

For restoring small anterior gaps you will need:

- 1 *BEGO Semados® Mini implant (including implant cover screw) (choice of Ø 2.9/ 3.1)*
- 2 *Mini open tray impression (including model analog), Article no. 56959*
- 3 *Mini healing posts (L3/ L4 mm)*
- 4 *Conical Wirobond® MI Mini posts (including prosthesis/technician screw), Article no. 56960*

QUICK x SOUND x COST-EFFICIENT = 100% HAPPY PATIENTS

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**BEGO SEMADOS® PREFABRICATED NON-PRECIOUS ALLOY BARS
– A CONCEPT FOR VARIABLE TREATMENT OPTIONS**

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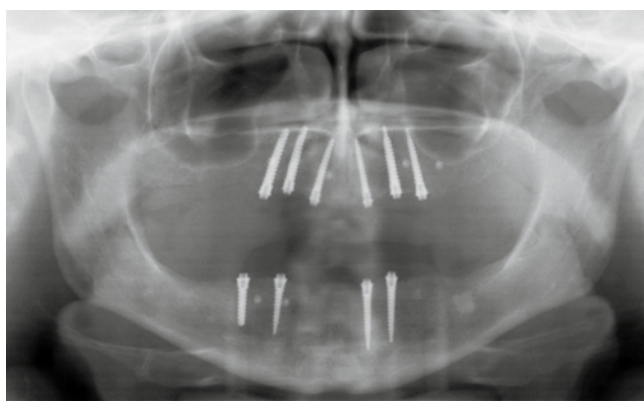
*Radiographic follow-up after osseointegration
(3 months)*

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**BEGO SEMADOS® PREFABRICATED NON-PRECIOUS ALLOY BARS –
A CONCEPT FOR VARIABLE TREATMENT OPTIONS**

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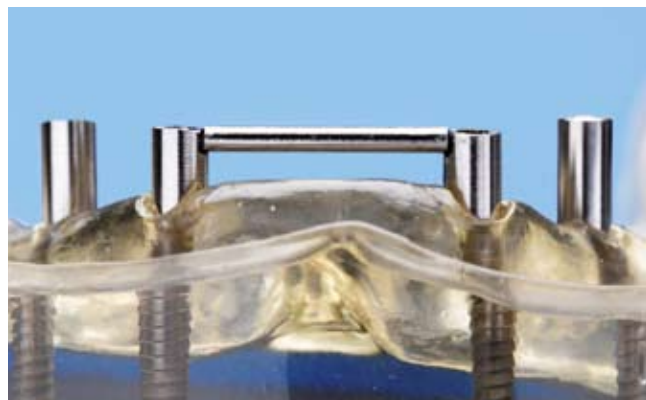
Impression taking with attached Wirobond® MI Mini bar abutments

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
Cutting to length the Wirobond® MI Mini bar abutments and the round bar on the model

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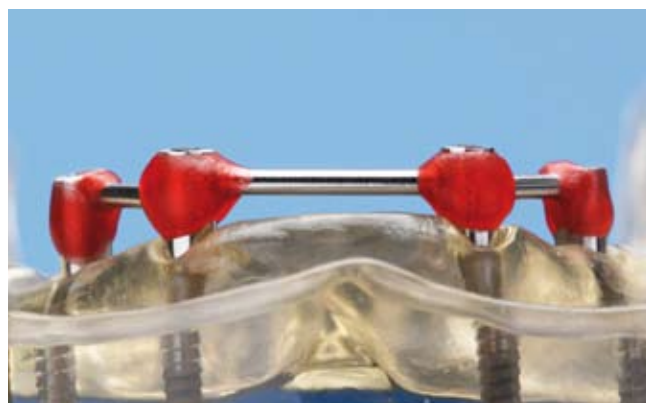
Fixation of the bar sleeves and segments with pattern resin

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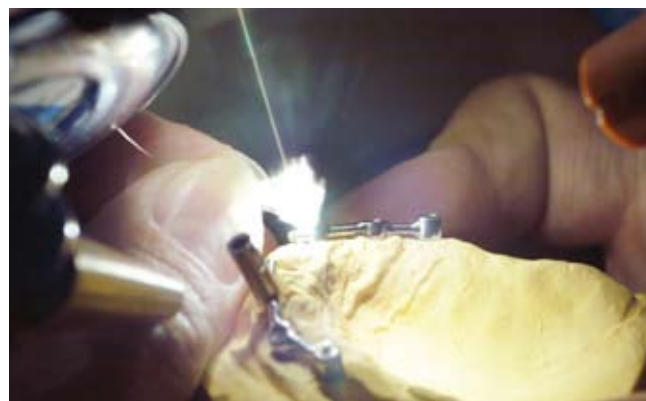Joining the fixed bar frame by laser welding

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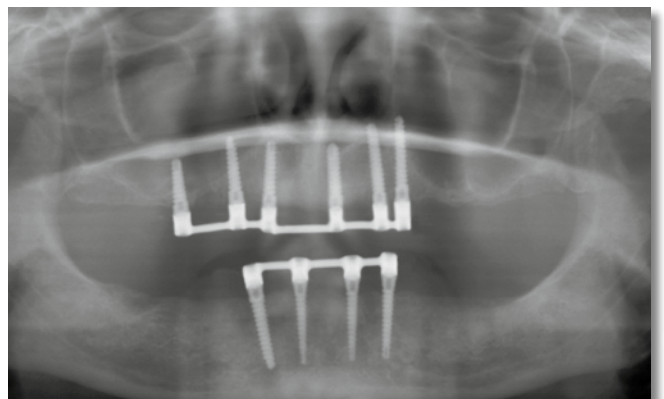
Radiologic follow-up of the case, 3 years after placement of the bar on BEGO Semados® Mini implants

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Prosthesis in the form of a cast base with integrated matrices

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BEGO SEMADOS® PREFABRICATED NON-PRECIOUS ALLOY BARS – SPECIAL FEATURES

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Immediate loading of the BEGO Semados® Mini implants and simultaneous attachment of the non-precious alloy prefabricated bars presents no problem

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*A proven technique is initial soft support with silicone-based lining material (soft loading)**

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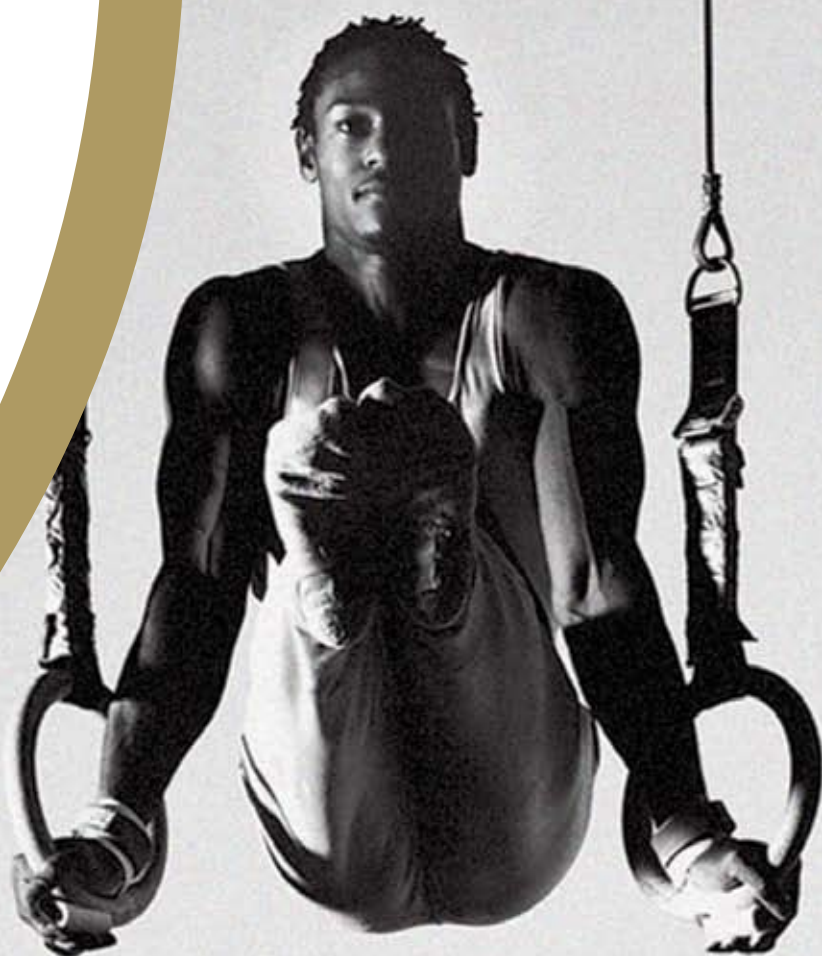
Simple integration of the matrices into existing dentures is possible and in many cases avoids the need to fabricate a new prosthesis

* BEGO Implant Systems GmbH & Co. KG recomienda el uso de Silagum-Comfort, de DMG (www.dmg-dental.com)

BEGO Implant Systems GmbH & Co. KG recommends Silagum-Comfort manufactured by DMG (www.dmg-dental.com)

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Mark the implant position with the initial drill

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Situation after removal of the healing posts

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*BEGO Semados® Mini implant
on which a restoration has been placed*

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*Screwing the Mini ball attachments onto the implants.
Placement and alignment of the female parts.*

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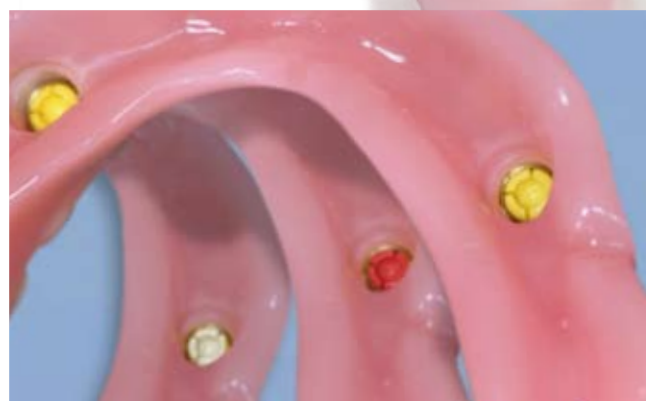
Complete the denture on the model using standard techniques. Optional intraoral completion (chairside) possible.

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The withdrawal force of the prosthesis can be adjusted using the three different female parts available (white for low retention – yellow for normal retention – red for high retention).

BEGO Implant Systems GmbH & Co. KG
Technologiepark Universität
Wilhelm-Herbst-Straße 1
28359 Bremen, Germany
Tel. +49 (0)421 2028-246
Fax +49 (0)421 2028-265
www.bego-implantology.com
E-mail: info@bego-implantology.com

BEGO IMPLANT SYSTEMS IBERIA, S.L.
Crta Reial, 105 bjs 8
08960 Sant Just Desvern
Barcelona
Tel. +34 933 720 325

BEGO
ACADEMIA
IMPLANTOLOGY

www.izi-online.de